

Dräger DrugTest® 5000 Drug Screening Device

No pipetting, no drips, no timing: Collecting an oral fluid sample is quick and easy with the Dräger DrugTest® 5000. The collected sample can then be analysed immediately for accurate results on the spot.



Benefits

Sampling and analysis in one system

The drug analysis system Dräger DrugTest 5000 consists of the sampling system Dräger DrugTest 5000 Test Kit and of the analyzer Dräger DrugTest 5000 Analyzer. This provides you with everything you need for a first assessment on the spot.

Ready for immediate use, if necessary

The Dräger DrugTest 5000 test kit provides you with a ready-for-use sampling system. You want to check a suspect person? Just remove the cover and hand the test cassette over to the test person.

Hygienic and painless: Collection without blood samples

The test person moves the top part of the collector briefly back and forth between his cheek and gums until the integrated indicator signals that enough of a sample has been collected. This usually takes less than a minute and never more than four minutes. The test person returns the cassette to you – and you will not come in contact with the saliva sample.

Blue means sample collection finished

The Dräger DrugTest 5000 automatically indicates the end of the sample collection process: If the collector has collected enough saliva, the indicator becomes blue and the sample collection process is finished. Afterwards you insert the test cassette and cartridge directly into the Dräger DrugTest 5000 Analyzer for evaluation.

Other advantages of the Dräger DrugTest 5000 test kit:

- high degree of conformity with blood samples and the detection limits valid in Germany (confirmed by a field test by the German Police University)
 - extremely low detection limit for THC as the main active ingredient of cannabis
 - clear results display
 - hygienic sampling and analysis
 - high level of acceptance among users and test persons
 - easy to use even with safety gloves
 - different configurations available (depending on model, several substances can be detected at the same time)
 - can be used at temperatures between 4 °C and 40 °C
 - can be stored in original package without any deterioration at up to 30 °C
-

Immediate evaluation of samples

Insert test cassette and cartridge, start evaluation, ready:

Benefits

The saliva sample can be evaluated immediately after sample collection. Just insert test cassette and cartridge into the analyzer – the evaluation will start immediately.

Easy and safe operation – even under adverse conditions:

Even when it becomes hectic: you control the analyzer with only three keys. The operation is menu-driven. Just follow the plain text instructions.

Quick and comfortable – results in just a few minutes:

Once the device is put into operation, the analysis will run automatically. This keeps your mind clear and your hands free for attending to your test persons. After only a few minutes, the result will be displayed.

Easy-to-read colour display:

The colour display is illuminated. Thus, you stay on top of the situation even under reduced visibility conditions. You can read the display text without any problems even from some distance or from an angle. The Dräger DrugTest 5000 Analyzer also displays the measuring results in clear text. Therefore, misinterpretations do not occur.

Evaluating samples wherever the situation demands:

Whether you use the sampling system with the 230 V power supply or the 12 V car adapter included, the permanently integrated, rechargeable battery makes you independent of any external power source and the analyzer is ready to use anytime and anywhere.

Storing 500 measuring results and managing them easily:

It stores the last 500 measuring results including date and time. You can enter further measurement details via keyboard or barcode reader. You can print the results with the Dräger Mobile Printer directly on the spot. To document the data, you can transmit them on the PC via USB.

4.5 kg lightweight for transportation:

The analyzer is easy to carry. For transporting the device and its accessories easily and safely, Dräger provides a matching carrying bag or a robust transport case.

Other advantages of the Dräger DrugTest 5000 Analyzer

- integrated self-test function: no semi-annual calibration necessary
- a variety of accessories (e. g. carrying bag, battery charger or compact keyboard)

The system detects a variety of substances such as:

- Amphetamines
- Benzodiazepines
- Delta-9-tetrahydrocannabinol (THC)
- Cocaine
- Methamphetamines
- Opiates

Benefits

- Methadone
- Ketamine

Dräger SSK 5000: Detecting drugs on surfaces or in substances

You want to know on the spot whether drug traces are detectable on a steering wheel or if the contents of a suspicious bag contain drugs? Easily answer such questions by taking a sample with the Dräger SSK 5000 drug sampler and transferring the specimen to the Dräger DrugTest 5000 Test Kit. Then you can analyse the sample with the Dräger DrugTest 5000 Analyzer.

Accessories



D-4814-2014

Test Kits

Ergonomically designed for easy handling and ready for immediate use, this innovative test kit consists of one major part, a test cassette complete with oral fluid collector.



ST-3451-2003

Compact keyboards

- with PS2 interface, dimensions approx. 28.2 x 13.2 x 2.4 cm.
- "QWERTZ" - German layout, REF: 8315095
- "QWERTY" - English layout, REF: 8315497
- "AZERTY" - French layout, REF: 8315142

Accessories

ST-8404-2006



Dräger Mobile Printer

The Dräger Mobile Printer prints breath alcohol and drug test measurement results without using a dedicated power supply. Operates on 4 AA batteries.

REF: 8319310

D-52608-2012



Carrying case for Dräger DrugTest 5000

REF: 8322675

D-4337-2014



Hard-shell carrying case

REF: 8319925

D-12909-2009



Barcode scanner

REF: AG02491

Related Products



Dräger DCD 5000

Simple, fast and secure detection of intoxicants and drugs: The Dräger DCD 5000 does not only enable an easy collection of saliva, you can also store the samples and send them to the laboratory for analysis – without the sample being falsified along the way.



Dräger Alcotest® 6820

Dräger Alcotest® 6820 is the logical successor to the successful Alcotest® 6810. The compact hand-held measuring instrument Alcotest® is quickly ready for use and is adaptable to international guidelines. With an even more sturdy enclosure and the proven Dräger sensor, Alcotest® 6820 is a reliable partner for breath alcohol analyses.



Dräger Alcotest® 7510

The Dräger Alcotest® 7510 is a compact and robust handheld breath alcohol measuring instrument, especially designed for advanced screening applications. Ideal for use by police as well as in commerce and industry as an alcohol screening device, it can also be used for evidential testing purposes, subject to relevant local and market sector approvals.

Technical Data

Operating voltage	12 V direct voltage (via supplied power pack)
Power input	normally 3 A
Dimensions (W × H × D)	200 × 260 × 250 mm
Weight	4.5 kg
Operating range	Operation: 4 °C to 40 °C Storage / transportation: -20 °C to +60 °C relative humidity 5 to 95 % (non-condensing)
Ports	optical for printer, PS/2, USB Slave
Time of measurement	depending on used test kit (typically < 9 minutes) optional: Analysis in < 5 minutes by increasing the THC cut-off to 25 ng/ml (after updating the device software)
Storage capacity	500 records or measurements

Ordering Information

	Packaging unit	Order No.
Dräger DrugTest® 5000 Analyzer	1 analyzer incl. power pack	83 19 900
Dräger DrugTest® 5000 Test Kit COC-OPI-BENZO-THC-AMP-MAMP*	20 pieces	83 19 830
Dräger DrugTest® 5000 Test Kit COC-OPI-THC-AMP-MAMP*	20 pieces	83 21 280
Dräger DrugTest® 5000 Test Kit COC-OPI*	20 pieces	83 19 990
Dräger DrugTest® 5000 Test Kit COC-OPI-BENZO-THC-AMP-MAMP-MTD*	20 pieces	83 23 156
Dräger DrugTest® 5000 Test Kit COC-OPI-BENZO-THC-AMP-MAMP-MTD-KET*	20 pieces	83 23 646**
Dräger SSK 5000	20 pieces	83 20 490
Compact keyboard, German keyboard layout ("QWERTZ")	1	83 15 095
Compact keyboard, English keyboard layout ("QWERTY")	1	83 15 497
Compact keyboard, French keyboard layout ("AZERTY")	1	83 15 142
Barcode reader	1	AG 02 491
Dräger Mobile Printer	1	83 19 310
Carrying case for DrugTest® 5000 Analyzer and accessories	1	83 22 675
Hard shell carrying case for DrugTest® 5000 Analyzer	1	83 19 925

* COC: cocaine – *OPI: opiates – *BENZO: benzodiazepines – *THC: cannabis – *AMP: amphetamine – *MAMP: methamphetamines – *MTD: methadone – *KET: Ketamine

** Sold only within the European Union

Notes

CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
www.draeger.com

MANUFACTURER

Dräger Safety AG & Co. KGaA
Revalstraße 1
23560 Lübeck, Germany

REGION EUROPE CENTRAL AND EUROPE NORTH

Dräger Safety AG & Co. KGaA
Revalstraße 1
23560 Lübeck, Germany
Tel +49 461 882 0
Fax +49 461 882 2080
info@draeger.com

REGION EUROPE SOUTH

Dräger Safety France SAS
25, rue Georges Besse
92182 Antony Cedex
Tel +33 1 46 11 56 00
Fax +33 1 40 96 97 20
europe.south@draeger.com

REGION MIDDLE EAST, AFRICA

Dräger Safety AG & Co. KGaA
Branch Office
P.O. Box 505108
Dubai, United Arab Emirates
Tel +971 4 4294 600
Fax +971 4 4294 699
contactuae@draeger.com

REGION ASIA PACIFIC

Draeger Safety Asia Pte Ltd
25 International Business Park
#04-20/21 German Centre
Singapore 609916
Tel +65 6308 9400
Fax +65 6308 9401
asia.pacific@draeger.com

REGION CENTRAL AND SOUTH AMERICA

Dräger Panama S. de R.L.
Complejo Business Park,
V tower, 10th floor
Panama City
Tel +507 377 9100
Fax +507 377 9130
contactcsa@draeger.com

Locate your Regional
Sales Representative at:
www.draeger.com/contact

